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GEKOFROST

Instructions d'utilisation - Instructions for use - V1 ed. 02/2026

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ECO-DESIGNED "RETURN TO ACTIVITY" KNEE SUPPORT + COMPRESSIVE COLD THERAPY

Description

The device is intended to provide knee support, pneumatic compression, and cold therapy for a single patient, within the scope of the indications listed below. The device includes:

- 2 removable articulated lateral stays (fastened with hook-and-loop straps).
- 1 main device consisting of 1 anterior cold pack and 1 posterior cold pack (with integrated Lycra lining).
- 1 manual pump + connecting tube.
- 1 additional jersey / skin interface fabric.
- 1 freezer bag.

Size: adult (user > 1m40 / 4'7").

Device characteristics and performance

This device combines wraparound cold packs, a pneumatic compression system (via manual pump), and removable articulated stays to provide optimal knee support while delivering the benefits of cryotherapy. The complete system helps to:

- Provide adjustable compression (by inflation) for a customized fit around the knee.
- Stabilize the joint during movement or immobilization (thanks to the lateral stays).
- Provide an analgesic (pain relief) and anti-edematous (swelling reduction) effect.

Indications

This device is intended to accompany, as medically prescribed:

- Pre-operative and post-operative knee immobilization.
- Rehabilitation with joint limitation.
- Ligament instability (cruciate, collateral...).
- Conservative or post-traumatic treatment of ligament, meniscal, tendon, or cartilage injuries.
- Immobilization of proximal tibia, condyle, or distal femur fractures.
- Other medical indications.

Contraindications Do not use in case of:

- Known allergy to any of the components.
- Unprotected wound or damaged skin in contact areas.
- Use outside the intended indications or intended size.
- Diabetes.

Do not use cold therapy in case of intolerance or history of reaction to cold, severe circulatory disorders (e.g., Raynaud's syndrome, arteritis), decreased skin sensitivity (risk of cold burns), medical contraindications to cold, or if the user is unable to properly communicate pain.

Warnings / Precautions

- Initial fitting: To be validated by a healthcare professional (in the event of a medical prescription).
- Pneumatic compression: Do not overinflate the device with the manual pump. Do not tighten the straps to the point of restricting circulation. Regularly check to ensure there are no excessive pressure zones.
- Cold therapy – essential rules: The device features an integrated Lycra lining that allows for direct application to the skin. However, if the cold feels too intense, use the additional jersey sleeve provided. Do not apply to irritated or damaged skin. Adhere to the prescribed duration and frequency: Use for 20-minute sessions, to be repeated according to the prescriber's instructions.
- Medical advice: In case of pain, swelling, abnormal sensation, inflammatory or allergic reaction, immediately stop using the medical device and consult your doctor, surgeon, or authorized healthcare professional (pharmacist, physical therapist, orthotist).
- Removable hinged splints: These are an integral part of the device. Their use depends on clinical evaluation and medical prescription; some protocols require their removal, while others require them to be kept in place.

Usage / Fitting

For fitting the knee brace and the packs, please refer to the recommendations and diagrams provided at the end of the manual.

- Cold preparation: Leave the packs in the freezer for at least 3 hours prior to use.
- Hinged splints (if required): Fasten the Velcro straps directly onto the brushed fabric part of the device (one splint on each side of the leg). Ensure the bend of the hinge is properly aligned with the knee.

Warning signs (immediate discontinuation)

Deflate the pump, remove the device, and consult a doctor in case of:

- Abnormal pain, tingling, numbness, loss of sensation.
- Persistent redness, cold burns, blisters, white/bluish areas.
- Cold, pale, blue knee/leg, increasing pain or circulatory discomfort.
- Increasing edema or significant discomfort. (Increased monitoring for at-risk patients: diabetes, circulatory disorders, fragile skin, elderly individuals).

Undesirable side effects

Possible: irritation/redness, chafing, circulatory restriction if over-inflated or overtightened, cold burns in case of misuse. Any serious incident that has occurred in relation to the device must be reported to the manufacturer and to the competent authority (ANSM in France: www.ansm.sante.fr).

Composition

41.28% recycled polyamide, 55.69% polyurethane, 3.03% polyamide. Adjustable hinges. Lining and Jersey: polyamide, elastane. Polycotton

Care and Cold Preparation

- Main device (Compressive cold packs): Hand wash with soapy water (or with a clean, damp cloth). Ensure the valve is tightly closed before cleaning the pack. Dry flat in the open air, away from any heat source. (Do not machine wash, bleach, or tumble dry). Never heat, puncture, or modify the device. Do not use if there is a leak.
- Hinged splints (removable): Remove before cleaning and wipe with a damp cloth.

In case of cooling gel leak

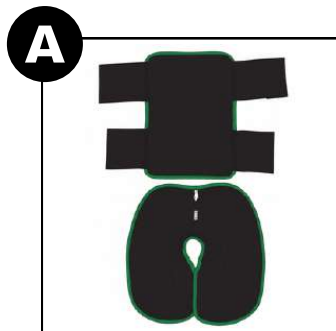
Do not ingest. Avoid contact with eyes and skin. Rinse thoroughly with water in case of contact. Handle with gloves; do not dispose of the gel in a drainage system or aquatic environment. In case of a leak, dispose of it with special industrial waste at a recycling center (do not release into the environment).

Storage

Store the device clean and dry, away from humidity and heat before its first use. Thereafter, keep the cold insert in the freezer after each use. Keep out of reach of children.

End of life and Recycling

This device is designed to be reused within a collection and refurbishment network. At the end of your treatment, do not throw it away: return the brace to your prescriber, your healthcare facility, or drop it off at a partner collection point (www.gekomed.com).

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© Gekomed